**UWAGA:**

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**ATTENTION:**

The brace should be used according to the instructions given by the doctor or physiotherapist.  
When used for the first time, the support should be put on by the experienced health-care personnel. Range of motion should be adjusted according to instructions given by the doctors or physiotherapist.  
Before the first use, please, read the user's manual carefully. If you feel discomfort while using the brace, in case of swelling or other skin problems, please, consult a doctor or physiotherapist.

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**INDICATIONS:**

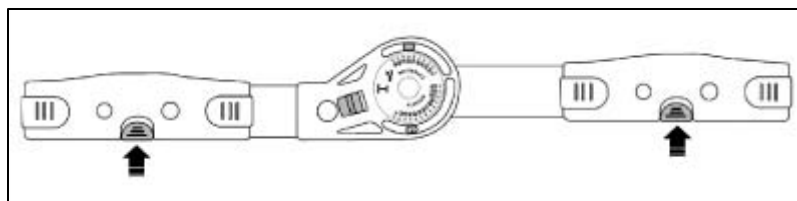
Conditions which require immobilization or control of knee joint range of motion after surgeries or injuries.

**CONTRAINDICATIONS:**

Conditions which require complete immobilization of lower limb intra-articular fractures, fractures of tibia, tibia and femur.


**USE:**

1. Adjust the length of lateral splints-press the button placed on the femoral and tibial part (marked in the picture) and expand them to desired length.

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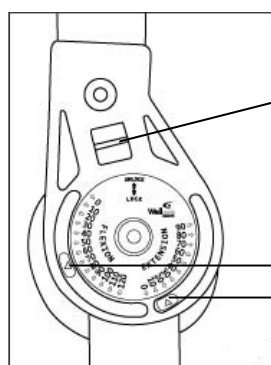
2. Required by therapy limitation of angular range of motion should be chosen and set before putting on the brace.
3. In order to set the angular range of motion, move the blockade placed on the hinge upwards to “open” position. Place the extension/flexion stops in appropriate position. Pull the blockade downwards. Remember to choose the same angular range of motion for both hinges.

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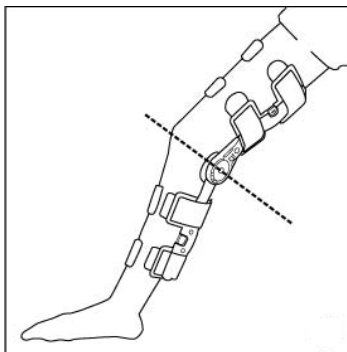
4. After adjusting all parameters put on the brace – remember to the position the hinge properly (the hinge axis should be little above articular space). The splints should be placed symmetrically on both sides of lower limb.

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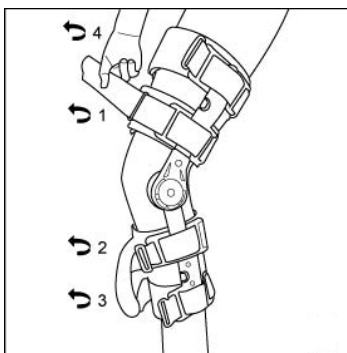5. After putting the brace on right place, fasten stabilizing bands starting from those close to the knee joint.

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#### Maintenance:

Padding elements and adjustment bands should be washed up by hand using mild detergents. Do not dry close to the artificial warmth source.

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#### ATTENTION

Not observing the rules of personal hygiene and washing the product with inappropriate detergents may result in skin abrasion or other discomfort related to perspiration and spread of bacteria. Please do not use the product together with warming ointment or gel. After washing, rinse the product carefully - residue detergent left in the product may cause skin vexation and damage of the orthosis fabric.

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