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VITEA®
CARE

User's manual
Padded commode chair
TYPE: fixed
DRVW021LA
Edition AM 08.2021

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**PLEASE READ WHOLE INSTRUCTION
CAREFULLY BEFORE USE.**

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Introduction

Thank you for choosing our product. We are deeply convinced that it will meet your expectations. Please read this document before first use. This manual is passed in order to provide the necessary information for the safe use of the device. The correct application of the recommendations will contribute significantly to extend the durability and aesthetics of the product. We want to emphasise that constantly we are trying to improve the quality of our products, which may result in minor changes that are not included in the manual

Product description

The padded commode chair allows height adjustment and quick and easy installation without tools. Available in non-folding version. The product consists of durable steel frame. The side frames are finished with soft armrests. The legs are finished with non-slip rubber feet. Included is a 5-litre bucket with a lid, and a toilet seat with cover. The product is easy to use and easily folded for storage or transport (folding version).

Product characteristics:

- plastic seat,
- steel frame,
- soft armrests,
- adjustable height,
- blue soft seat,
- 5-litre bucket with a lid.

Application

The product is helpful in care of patients forced to stay in bed or using wheelchair. It allows for relieving one's physiological needs in seated position. It helps with patient care. A chair without a bucket can be placed over a toilet bowl.

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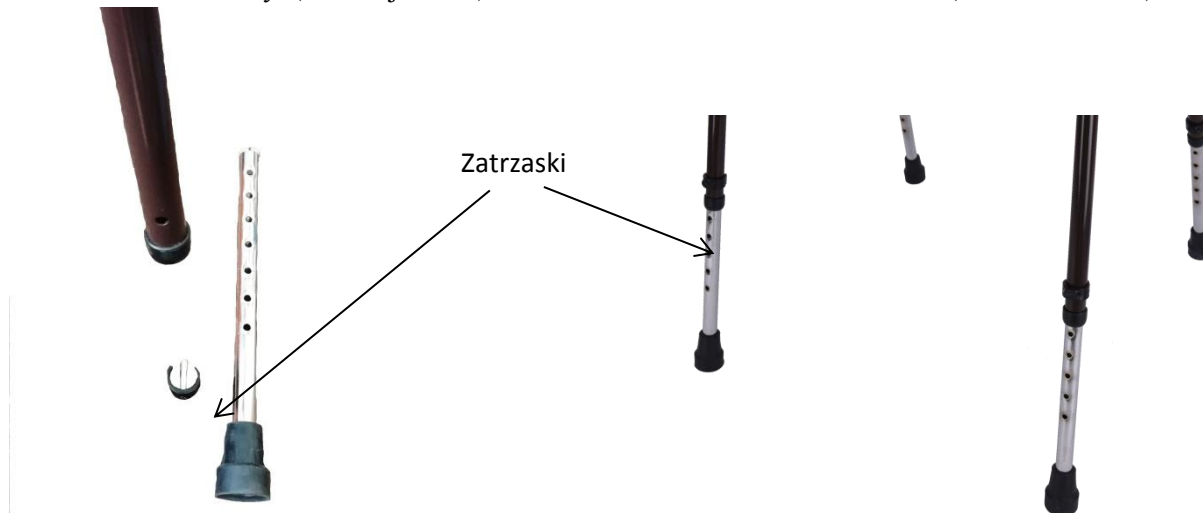
Contraindications

Contraindications to a sitting position. Patients with impaired balance and coordination, as well as intellectually disabled patient should not be left unattended when using the product.

Use

Padded commode chair

TYPE: fixed

- Remove the product from the packaging.
- To the chair frame should be mount four feet by pressing the latches on the legs and sliding the feet in the chair frame. The latch should appear in one of the holes of the frame. (Illustration 1., 2).

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- The seat with a hole should be mounted in the back of the chair.
- The seat is mounted by using clips.

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Safety of use

WARNING !

- Before using the cushion, refer to this manual and keep it for future reference.
- The commode chair is designed to be used on flat, straight surfaces.
- The use of commode chair on the rough surface may result in falling down and injury.
- It is strictly forbidden to stand on the commode chair.
- Maximum load of the commode chair is 150 kg.
- When using this product for the first

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time, assess the risk and suitability of a given person using the product.

- g) Before use, always make sure the chair is stable and the legs are on the same level. Make sure that all sides of the frame are completely locked in a stable position and the toilet cover and seat are firmly attached to the rear frame.
- h) Regularly check the equipment for damage and wear. If the equipment is damaged, stop using it.
- i) Keep away from direct sunlight and heat source.

Everyday use and maintenance

1. After each use should be emptied and disinfect container and a toilet seat.
2. Use the toilet detergents according to the instructions given by a manufacturer,
3. Clean the shower stool frame with a moist cloth.
4. Do not use sharp cleaning tools (risk of damaging the protective cover).
5. Clean with gentle detergent.
6. Regularly clean the commode chair mobile elements (wheels).

Keeping all the elements clean is essential for preserving the product working parameters. Keeping the commode chair for a long time in a very moist room or outdoor may result in loss of esthetic and functional parameters of the product.

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Weight: 8.5 kg

Materials: steel, plastic.

Serial number/ production date: on the product

Product durability: 2 years from production

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Repair

- In case of reparation, please contact the producer.

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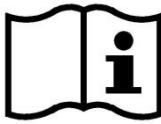
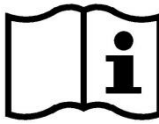
Customer service

- If you have any questions or need help, contact the producer.

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