INSTRUCTIONS FOR USE

Wall shower bench with legs, foldable

Product code: JMC-C 5105

1. Characteristics

Shower bench consists of seat made of durable PVC, height adjustable legs with anti-sliding rubber tips and mounting plates for attaching to the wall. The foldable design saves space when the product is not in use. Owing to the push button height adjustment, the bench can be adjusted to patient's height. The bench is made of durable materials resistant to humidity.


The set includes:

Bench and installation kit (wall plugs with hex key).

Note! The kit contains only the standard type of wall plugs. Before installation, make sure in what substrate the installation will take place and choose the right type of wall plugs accordingly (e.g. brick, hollow block, plasterboard)

Elements of the bench:

- a. mounting plates
- b. mounting holes
- c. seat
- d. height adjustment – locking pin
- e. anti-sliding tips

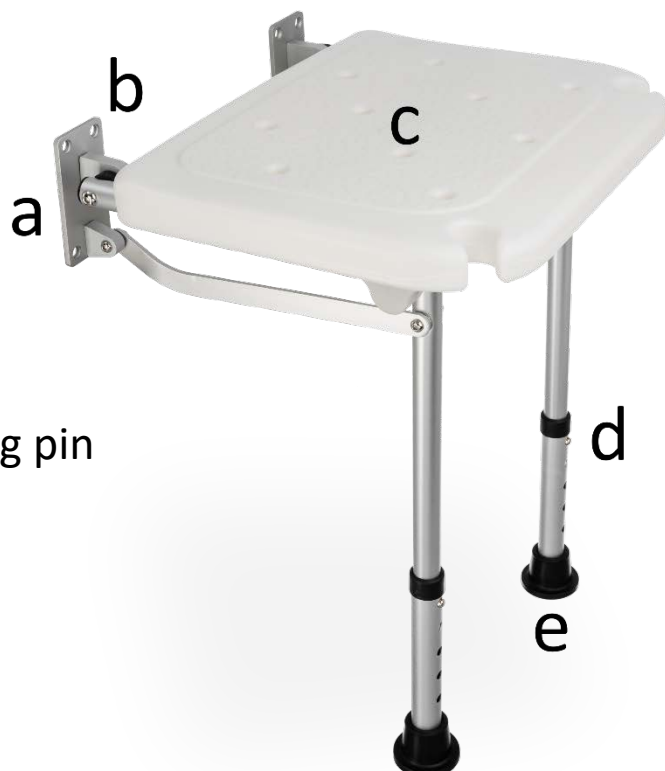2. Application

The shower bench is ancillary equipment which helps the user to perform hygienic activities. The chair is recommended to people with reduced mobility which prevents or hinders the unassisted performance of daily living activities. The bench can be used in the shower, in bathrooms and other humid rooms at home and in hospitals.

3. Assembly

Step 1. Before assembly, adjust the bench to user's height. To adjust the height, press locking pin, select appropriate height, and lock the position. The legs are adjustable in 2.5 cm increments.

Note! The pins must be set on the same level in each leg.

Step 2. Mark the places where the mounting plates will be fixed on the wall. Mark the positions of the 8 mounting holes.

Step 3. Drill 8 pre-marked mounting holes in the wall.

Step 4. Screw the mounting plates to the wall using 8 appropriately selected wall plugs - selection of plugs depending on the substrate (brick, hollow block, plasterboard)

Note! Do not use a bench if the legs do not have a stable and firm support on the ground.

4. Technical data

Seat dimensions	32-40 cm
Height adjustment	42-52 cm
Weight	2,00 kg
Maximum capacity	136 kg

5. Notes

Timago International Group does not bear responsibility of improper use of the wall shower bench, failure to comply safety regulations and misuse.

6. Maintenance

- Clean the shower chair regularly using only water, sponge and mild detergents for plastic materials such as soap. Dry thoroughly.
- **Do not use** for cleaning: strong detergents, abrasive agents, bleach, petrol or solvents (it may cause frame damage, visible defects and in consequence - loss of warranty rights).
- Systematically please check:
 - tightening of screws
 - condition of locking pins
 - condition of tubes (cracks etc.)

It is recommended to check the rubber tips regularly. In case they wear-out, replace the tips with new ones.

7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained

9. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date

	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors.
 Learn more at: www.timago.com. Thank you for choosing Timago!

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