WHO – STAB PLUS

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WHO-STAB PLUS is a closed orthosis with thumb hold of wide therapeutic use. It helps treatment and rehabilitation of wrist injuries and dysfunctions. Its construction ensures three-point stabilisation and maintains the thumb in functional position. Due to the profiled liners it allows to immobilize the wrist in a chosen therapeutic position. Four additional bands allows to adjust the WHO-STAB to the patient's hand and soft lining ensures comfort of use.

Indications:

Treatment and follow-up treatment of wrist, thumb and lower forearm injuries, inflammation and other dysfunctions; injuries to medial and radial circumerential nerves.

Follow-up treatment after fractures and while taking up sport activities after convalescence.

Product description:

WHO-STAB PLUS is made of velour knitwear. It is equipped with four Velcro bands which allows to adjust the pressure to the individual needs and ensures excellent thumb stabilization. The construction consists of three liners: two profiled liners, one aluminium and one elastic liner made of PCV. WHO-STAB PLUS is available in two versions: right and left.

Putting on:

Loosen the bands and place the liners in the tunnels (in case they are not already placed).

Put the orthosis on a hand.

Fasten the bands and adjust the pressure.

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ATTENTION

Not observing the rules of personal hygiene and washing the product with inappropriate detergents may result in skin abrasion or other discomfort related to perspiration and spread of bacteria. Please do not use the product together with warming ointment or gel. After washing, rinse the product carefully - residue detergent left in the product may cause skin vexation and damage of the orthosis fabric.

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