UWAGA

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ATTENTION!

The final decision concerning time/ways of using the equipment belongs to a medical specialist.

Do not take off the brace without consulting your doctor. If you notice any irritation, excoriation or if you experience any discomfort while using the brace, please, consult a doctor/physiotherapist.

Special attention should be paid in case of using the brace by patients with diabetes, circulation disorders or neuropathy.

Do not remove foam padding – it is directly attached to brace's frame.

Foot and shin brace has wide therapeutic use. It effectively helps rehabilitation and treatment in case of injuries, dysfunctions of ankle joint and foot. Its construction provides optimal stabilization and enables regulation of dorsiflexion / plantarflexion movement. Five additional bands provide optimal adjustment and comfort of use.

Indications:

Fractures/cracks of metatarsal bones or in post-operative treatment
Cracks of heel/tarsus/ great toe bones.
Surgeries of great toe joint.
Soft tissue injury, injuries of calcaneal tendon.
Sprain/dislocation of ankle joint.
Protection and post-traumatic stabilization of foot and ankle joint area.
Immobilization of ankle joint according to chosen therapeutic range.
Post-operative treatment.

Contraindications:

Conditions which require complete immobilization.
Open, unstable fractures.
Difficult to heal wounds, sores and other skin changes.

Product description

The brace is equipped with velour knitted-fabric padding with foam. There are also five additional Velcro bands which allow to adjust compression to individual needs. Two splints provide lateral stabilization. The hinge allows to regulate the dorsiflexion / plantarflexion (0° - 45°) every 15°. One, universal model for left and right limb. Anatomically moulded outsole enable stabile, similar to regular walk.

Use:

- Unfasten the bands and remove the padding.
- Place the limb inside the padding and fasten the padding so that it would adhere to the limb.
- Choose the required angular range of motion – place the bolt in slot marked with appropriate flexion/extension degree. Always make sure that the setting of both hinges is the same.
- Gently half-open the brace's frame and put the limb with padding.
- After putting on, make sure that the brace's frame covers stably lateral and medial ankle and that the lateral elements of the frame are placed centrally on the lateral plane of the limb and provide optimal comfort of use.
- Fasten the bands – begin from those placed on the foot.
- In order to take off the brace, unfasten the bands, padding and take out the limb.

Maintenance:

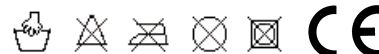
Remove the padding from the brace's frame and wash up by hand using mild detergents. Do not machine wash. Do not chlorinate, do not whirlpool, wring gently before drying. Do not use a brush. Dry in horizontal position in distance of artificial sources of heat.

The frame should be cleaned with a moist cloth using mild detergents.

ATTENTION

Not observing the rules of personal hygiene and washing the product with inappropriate detergents may result in skin abrasion or other discomfort related to perspiration and spread of bacteria. Please do not use the product together with warming ointment or gel. After washing, rinse the product carefully - residue detergent left in the product may cause skin vexation and damage of the orthosis fabric.

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